

Staebel Slideezi Support Straps - Pair

Code: 210064



Our range of Staebel Slideezi Support Straps are used to reduce the risk of over-stretching and reaching in frequent supine transfers. Attaching straps to all transfer devices with handles.

USER INSTRUCTIONS



1. Unclip the buckle and thread the open loop through the handle of the Slide Sheet or patient handling device.



2. Clip the buckle back into place. The Staebel Slideezi Support Strap are now securely in place.



3. Take a secure hold on the strap, adopt a safe and stable posture and work in a co-ordinated manner with a colleague for a safe transfer.



4. Transfer the patient in a safe manner in accordance with your organisational policies or procedures.



5. Once the patient has been securely transferred, unclip the strap from the device, clean and store.



6. For Economy Extension Straps without a buckle (Product Codes: ES40, ES60, ES100), there are two loops. Simply pass one of the loops through the handle of a product which is used to perform the transfer (Flat Sheet with Handles / Transfer Board etc) and then pass it through the other strap loop and pull through. The strap will now be securely looped around the transfer device handle.

SLIDEEZI SUPPORT STRAPS RANGE

EXTENSION STRAPS – WIPECLEAN

No need to launder reducing the chance of losses. A new concept in extension straps, these can be wiped down and/or washed in soapy water, rinsed and then left to dry. Used to reduce the risk of over-stretching and reaching in frequent supine transfers, they simply clip onto the handles of patient handling equipment. They can be kept near your Patslide so that they are always available when needed.



EXTENSION STRAPS – DELUXE

Supine transfers from a bed to a trolley and vice versa are easy with these Extension Straps – Deluxe. Staff have often injured themselves by overreaching or overstretching when repositioning patients. But not anymore. These straps make patient transfers easier and safer for carers and patients don't feel that they are being pushed and pulled. They can be attached to a lateral transfer board to safely slide a patient from one surface to another.



ECONOMY EXTENSION STRAPS

Wonderfully simple, these flat webbing straps have a handle at each end and can be used to help supine transfers from bed to bed or bed to trolley etc. You will love how simple they are to attach; just pass one of the loops through a handle of a flat slide sheet or similar and then pass it through the other looped handle and pull tight. The strap is secure. Sold in pairs.



Specifications

Extension Straps – Wipeclean (pair)	750 x 30 mm	210064
Protturn™ Straps – Wipeclean (3 in pack)	1700 x 30 mm	210289
Extension Straps – Deluxe (pair)	750 x 30 mm	210063
Economy Extension Straps (pair)	400 mm	N/A
Economy Extension Straps (pair)	600 mm	N/A
Economy Extension Straps (pair)	1000 mm	N/A

CLEANING INSTRUCTIONS

- Wash as per laundry instructions.
- Can be wiped clean, between use with the same patient, with natural water and detergent or wipes that don't contain alcohol, solvents, bleaching or abrasive agents.
- Cleaning materials used should be patient safe and biodegradable.

DISPOSAL INSTRUCTIONS

- Non-recyclable.
- Dispose of as clinical waste if it has been used by an infectious person.
- Otherwise dispose of through normal waste management.