

Staebel Patient Transfer User Manual

Product Code: 212145



Information before using the product / the functioning of medical products

1. Please read the enclosed instructions for use carefully.
2. Use the product only for the intended purpose of use.
3. Product management should be allowed only for the personnel familiar with the functioning of the product.
4. The maintenance as well as the safety and technical control must be carried out only by an authorised company or by any other professional company authorised by the Staebel company.
5. Assembly, extensions, revisions or repairs must be carried out only by an authorised company or by any other professional company authorised by the Staebel company.
6. In case of any technical interventions in the sense of additions or revisions as regards the products that are not carried out by Staebel or any other professional company authorised by Staebel, any guarantee in relation to revisions as well as in relation to the product and its functions referring to such revision, shall be abolished.

Preventive measures

Please read the instructions for use carefully. Use the product only for the intended purpose.

The remarks under the heading »WARNING« must be taken into consideration.

Purpose of use

This Patient Transfer is used for helping the patients when standing up from the bed, toilet, chair or wheelchair and for transporting of the patient. It is intended for patients who can stand, but not necessarily walk.

Regulations and norms

Provided that certain norms or regulations are valid, they should be taken into account.

CE mark

CE-mark refers to the guideline 93/42/EWG for medicinal products.

Responsibility and guarantee

The manufacturer of the product is responsible for the safety and reliability of the product only if:

- A. the product is used solely in the sense of the purpose of use.
- B. the controls over the functions are carried out on a regular basis.

Transport damages

Upon delivery, please check if the product or the packaging is damaged in any way. In this relation, the declaration date should be taken into consideration: there is a 2-day deadline for visible damages and a 7-day deadline for hidden damages.

Serial number

Serial number can be found on the lower frame. Please state the serial number at enquiries.

Product display – Staebel Patient Transfer

1. Handles
2. Handle for flexible base
3. Tight support
4. Knee support
5. Wheels with brake
6. Plate stand
7. Wheels without brake

Specifications

Product Code	212145
Width	640 mm
Length	930 mm
Height	1050 mm
Inside Base Width	450 - 830 mm
Outside Base Width	580 - 970 mm
Weight	33 kg
SWL	180 kg



Product management

The Patient Transfer is ready to be used right after removing the packaging.

Use of Patient Transfer:

Open both tight supports by pulling them with hand on the side and upwards.



Step by step use



WARNING

When accommodating the patient the wheels must be locked. (See Fig. 1)

To avoid damage on the equipment always close the padded seats. (See Fig. 2)



Fig. 1



Fig. 2

Maintenance

The product should be wiped with a duster. Approximately every two months the framework should be polished with a polishing paste. Knee and thigh supports are made of a material that is easy to maintain – composition leather, tested according to DIN 54020. Such coating may be wiped with a dry or damp cloth; detergent for sensitive laundry can also be used if necessary but it should be wiped straight after it has been used (the stains should be removed as soon as possible in order to prevent the absorption in the material). Finally, the material should be cleaned with warm and clean water.

Ingrained dirt can be removed if rubbed with a cloth, moistened with alcohol (the cloth should not be wet). Cleaning agents that contain fats or oils should not be used. Similarly, solvents (like acetone) or rough cleaning agents should not be used.

Safety and technical control

In accordance with »Medicinal Products Act« the safety and technical control should be carried out once a year.

Repair, servicing and spare parts

In case of any technical problems, please contact:

Staebel Products | 1800 772 287 | www.staebel.com.au | hello@staebel.com.au