

# Staebel Fixed Height Treatment Table User Manual

Product Code: 111080



## Thank you for choosing the Staebel Fixed Height Treatment Table.

Assembly of your new free standing table is very simple and tool free. Please see the following steps.

1. Remove all bubble wrap and ties from the table to leave the legs and the table exposed
2. Stand over the table. Hold one side of the table legs and pull them slowly upward until they are vertical.
3. Slowly lower the table legs into the brackets on each corner of the table
4. Line up the pin and the groove then push into place. Put extra force onto the leg bars once in position so it slots in tightly and deep into the groove.
5. Repeat on the other end of the table.
6. Turn the table over onto its legs and give an extra push on each end of the table once free standing to make sure it's locked in the right place.
7. For the head section insert the metal u shaped bar into the grooves on the back of the head section to the desired angle. Remove the u shaped bar from the grooves completely to lower it to a flat position.

## TABLE LEG INSTALLATION



The table is shipped completely assembled (Figure 1)



(Figure 2)




(Figure 3)



(Figure 4)

Lift the table leg and line up the pin into the groove. Then push onto the leg bars once in position so it slots in tight & deep into groove.

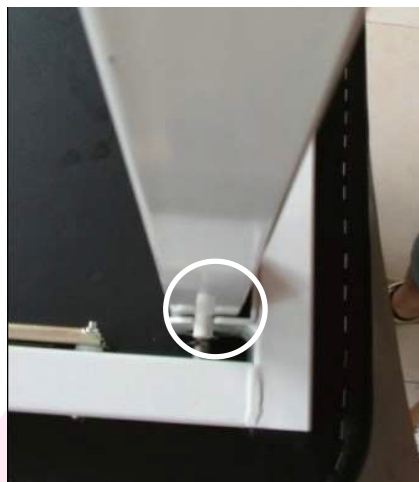

(Figure 5)

Turn the table over on it's legs and give an extra push on each end of the table once free standing to make sure it is locked in the right place.



(Figure 6)