

Staebel Gynecology & Urology Chair User Manual

Product Code: 71160



Thank you for choosing the Staebel Gynecology & Urology Chair.

1. Before use carefully read the enclosed instructions for use.
2. Use the product only for its intended purpose.
3. Product can be used only by personnel who are familiar with the functioning of the product.
4. The maintenance, as well as the safety and technical check-ups, can be carried out only by an authorised service company or by any other professional company authorised by Staebel.
5. Assembly, revisions or repairs can be carried out only by an authorised service company or by any other professional company authorised by Staebel.
6. If technical modifications, including upgrades or revisions of the products, are not carried out by Staebel or any other professional service company authorised by Staebel, any guarantee for such revisions, as well as for products on which such revisions were carried out, shall be deemed void.

Preventive Measures

Carefully read the instructions for use. Use the product only for its intended purpose. Always follow the notes under the »**WARNING**« signs.

Intended use

Gynaecological examination chair is designed to support a woman's body in the appropriate positions during gynaecological examinations, e.g., colposcopy, and other procedures. Back section and seat section are height and angle adjustable. Chair includes adjustable support handles and adjustable leg support mounted to the EURO rails of the chair. It enables easy changing of positions with regard to the patient's and doctor's comfort. Gynaecological examination chair is designed especially for hospitals and other healthcare settings.

Regulations and norms

- Always ensure compliance with valid norms or regulations.

CE Mark

- CE mark refers to the Directive 93/42/EEC for medicinal products and MDR 2017/745.

Responsibility and guarantee

The manufacturer of the product is responsible for the safety and reliability of the product provided that:

1. The product is used solely for its intended use.
2. The functional check-ups are carried out on a regular basis.

Transport damages

Upon delivery, please, check the product and its packaging for damages. Always refer to the delivery date: visible damages should be reported within 2 days after delivery and hidden damages within 7 days after delivery.

Serial number

The serial number is located on the product label, which can be found on the inner side of the product left leg.

PARTS DESIGNATION

1. Head section
2. Back section
3. Seat section
4. Stainless steel bowl
5. Electric linear actuator
6. EURO rails
7. Paper roll holder
8. Retractable caster system



Product Description

The 700 mm wide operating surface is covered with washable Skai. It is divided into a fixed angle head section, a back section and a seat section with an opening. The length of the chair top surface is 1380 mm. All electric actuators are operated manually and the positions can be changed even if the table is fully loaded.

The chair is fully equipped by EURO rails around the leg segment, it has adjustable support handles, adjustable leg support, paper roll holder and removable stainless steel bowl. All elements can be easily and comfortably adjusted and removed from the chair, which means that the patient can also get on and off the chair from the side.

Specifications

Product Code	71160	Head Section Inclination	12°
Overall Width	700 mm	Back Section Inclination	-10° to +65°
Overall Length	1380 mm	Seat Section Inclination	0° to +30°
Head Section Length	300 mm	Height	560 - 940 mm
Back Section Length	600 mm	Weight	74.5 kg
Seat Section Length	470 mm	SWL	150 kg

First Use

After removing the packaging install adjustable support handles and adjustable leg support by sliding it on the EURO rails one by one. Set the desired height of each part and tighten it with two side screws. The Gynaecological examination chair is ready for use.

Product Use

The Gynaecological examination chair height, back section position and seat section position are adjustable with handset. Handset allows raising/lowering and tilt of the back section and seat sections. The back section position can be set with UP/DOWN arrows in the handset first row. The height of chair top can be adjusted by pressing UP/DOWN arrows in the handset second row. The seat section position can be set with UP/DOWN arrows in the handset third row.

The handset also includes three memory functions: starting position, resting position and chair flat top position.



To avoid injuries, make sure that the patient is never left unsupervised.

- To prevent the device from tipping do not raise or lower other equipment close to it.
- To prevent cross-contamination disinfect regularly between each use, if necessary, or daily, as a minimum.
- To avoid injuries of the patient and caregiver never modify the equipment or use incompatible parts.

Maintenance

The product is recommended to be cleaned and/or disinfected regularly between each use, if necessary, or daily, as a minimum. Clean the product with a damp cloth using warm water and a disinfectant cleaner. Disinfectant wipes, which are pre-soaked with a 70% (v/v) solution of isopropyl alcohol, can also be used. Head, back and seat section are made of composition leather, which is easy to maintain and complies with DIN 54020. This type of coating can be wiped with a dry or damp cloth; detergent for sensitive laundry can also be used, if necessary, but it must be wiped immediately after use (the stains should be removed as soon as possible to prevent its absorption by the material). Finally, the material should be wiped with warm and clean water.

Safety and technical checks

Pursuant to the "Medical Product Act" the safety and technical checks should be carried out once a year.

Repairs, service and spare parts

In case of technical problems, please, contact Staebel or your local Staebel representative.